
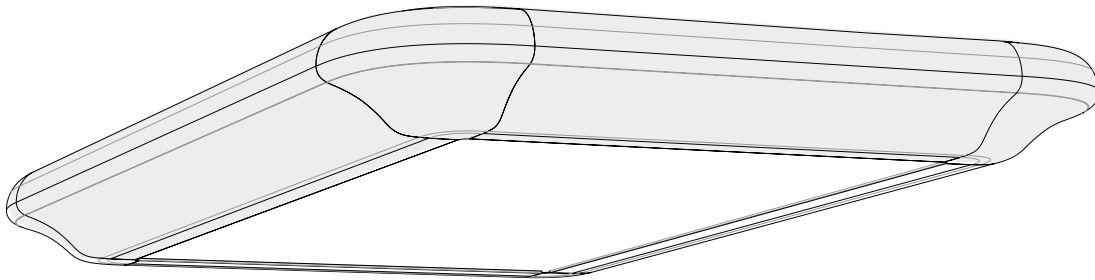


Denta / Denta plus
Owners Manual

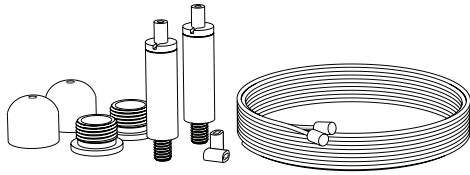





Note: Don't use sharp objects when unpacking the luminaire!

1. Unpacking Your Denta / Denta plus

Carefully open the packaging without using any sharp objects!

In the packaging you will find the following items:

- Wire arrangements for attaching the luminaire to the ceiling. (4 pcs)
- The Denta / Denta plus luminaire

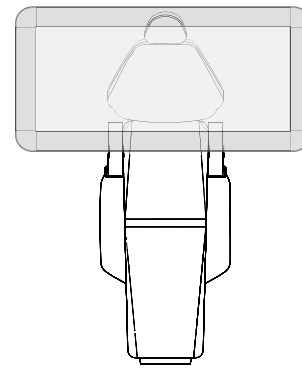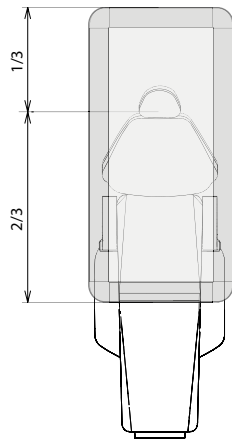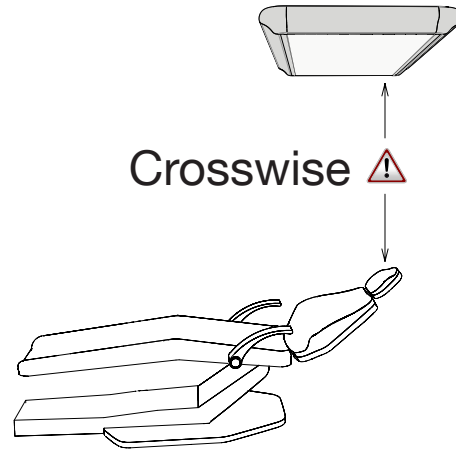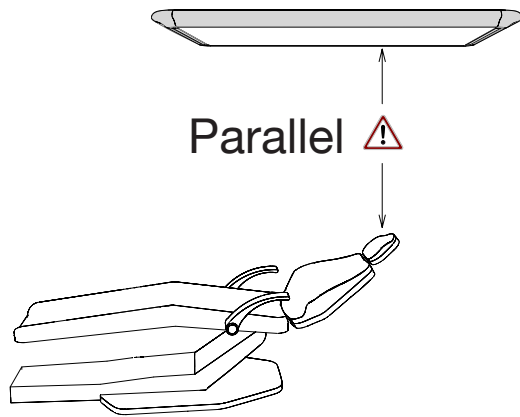







2. Positioning Your Denta / Denta plus luminaire

The luminaire should be positioned in one of these two positions:

- Parallel position. The eyes of the patient should be in line with 1/3rd of the luminaire.
- Crosswise position. The eyes of the patient should be in line with the top lightsource.

Note: The height of the luminaire should be 2,2 m above the floor level.


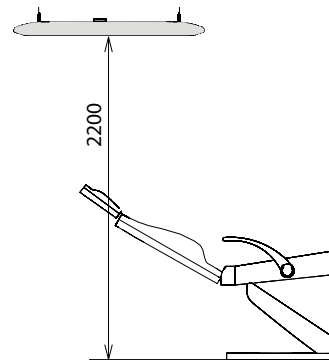


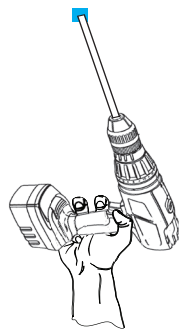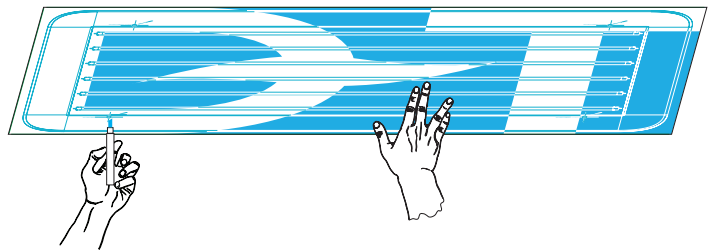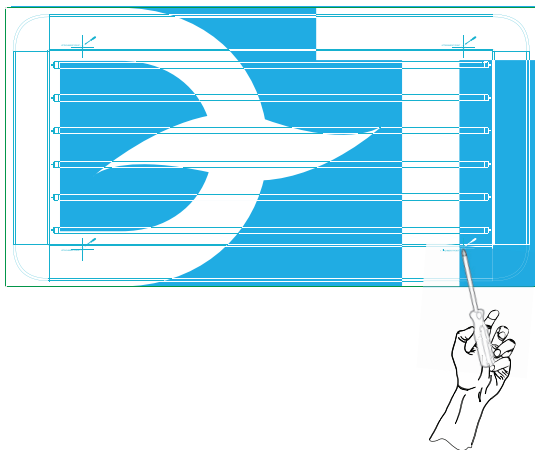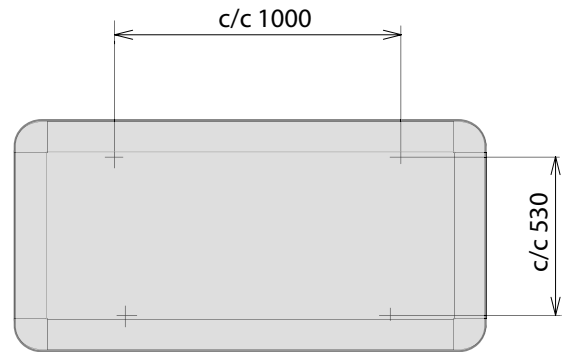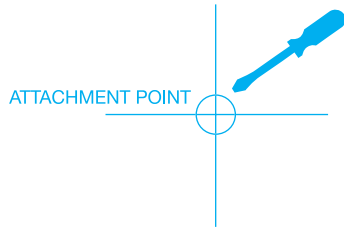

3. Preparing the mounting of Your Denta / Denta plus luminaire



AFTER unpacking the luminaire you can use the packaging as a guide for marking the attachment points in the ceiling.

- Use the front part of the packaging and punch four holes with a screwdriver or similar.
- Position the cardboard lid where you want to mount the luminaire, mark out the four points and drill your holes for the attachment.

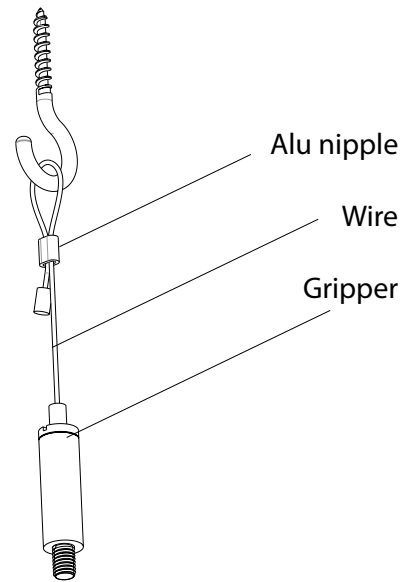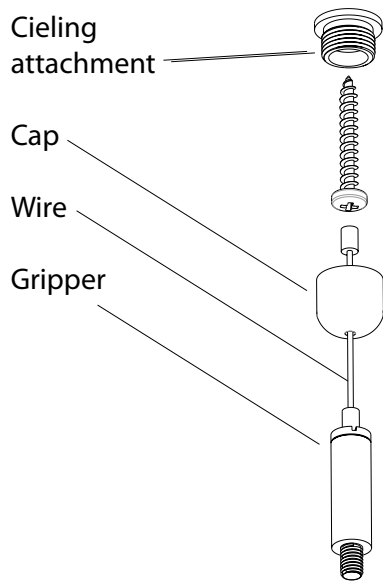

4. Mounting Your Denta / Denta plus luminaire in the cieling

These items are provided with the luminaire. Use the one suitable for your needs.

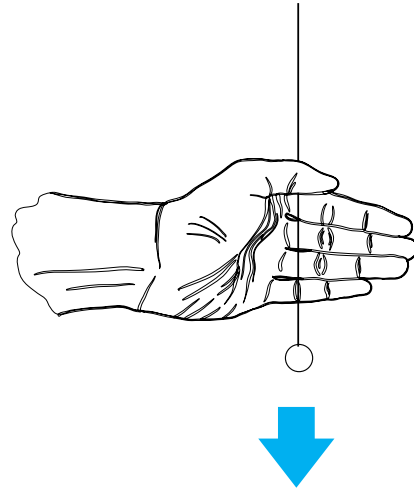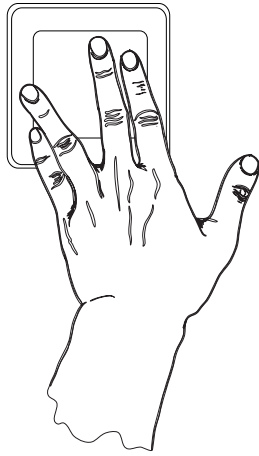


Caution! Fade out the lamp when working with light-curing materials.

5. Operating Your Denta luminaire

The luminaire is normally operated either by a wall switch or by the pulling string. The luminaire could be faded in such a way that the two middle sections of lights are shut off by a pull of the string.

The indirect light (upward lighting) is optional.

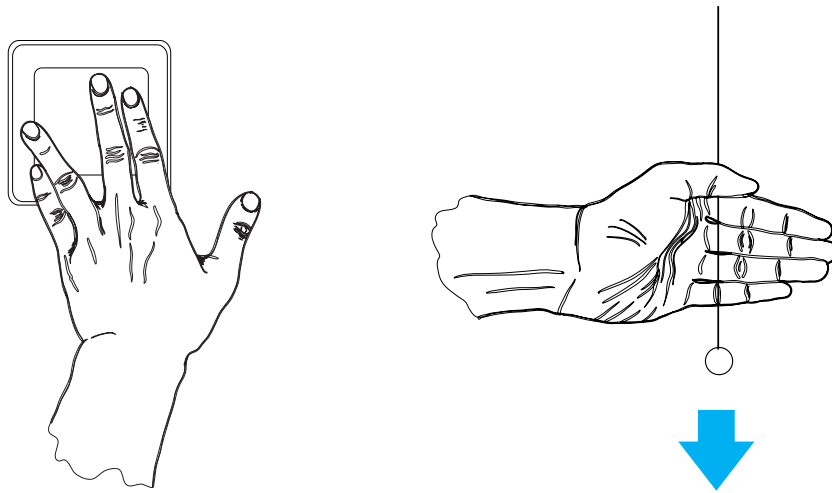


Caution! Fade out the lamp when working with light-curing materials.

6. Operating Your Denta Plus luminaire


The luminaire is normally operated either by a wall switch or by the pulling string. The dimmer function is operated by the pulling string as indicated below. The last dimmer setting is remembered the next time the luminaire is switched on. The indirect light (upward lighting) is optional.

- Switching the lamp an/off: short pull (<0,5s)
- Dimming: Long pull (>0,5s), the dimming direction is changed with each pull.

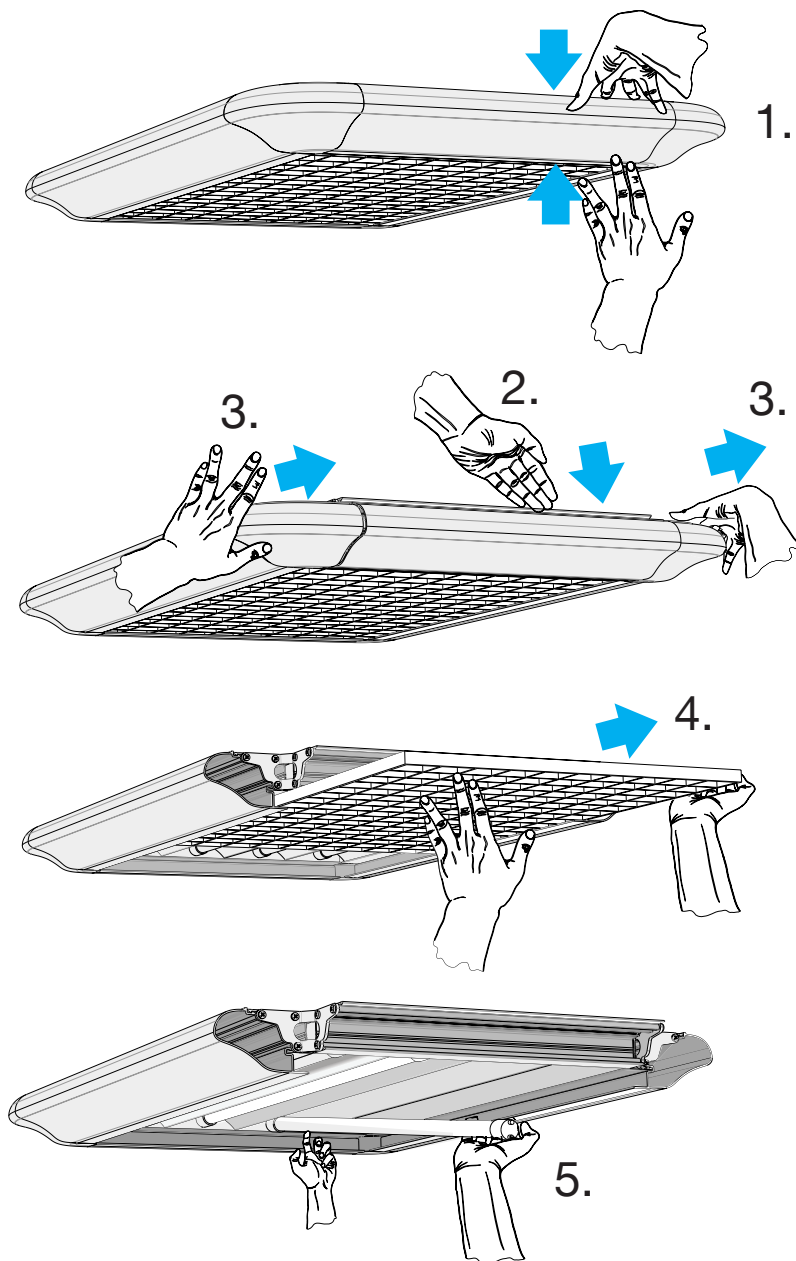

7. Exchange of lights

- Denta: 6 x 28W
- Denta plus: 6 x 54W

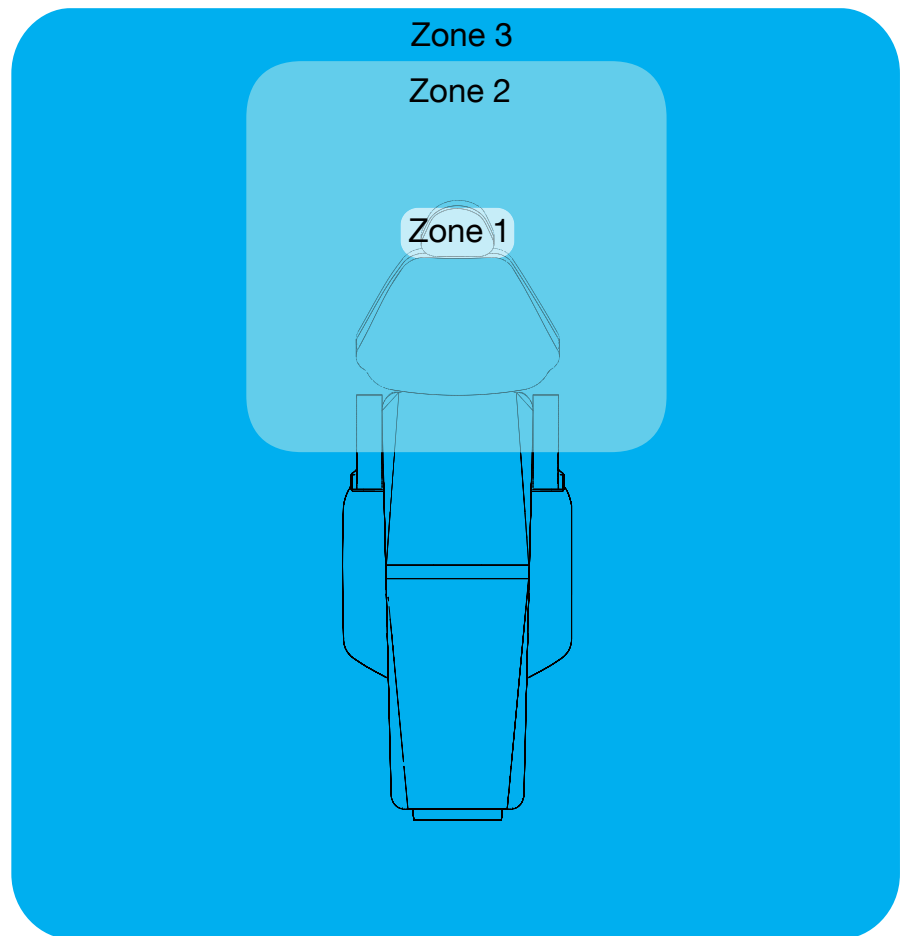


8. General Information

Image of preferred light conditions for dentists.

- Zone 1 = 20 - 40 000 LUX
- Zone 2 = 3 - 5000 LUX
- Zone 3 = 500 - 1000 LUX

On-line support:
<http://d-tec.se/support>

- Bruksanvisning
- Manuel d'utilisation
- Bedienungsanleitung
- Manual del propietario
- Käyttöopas

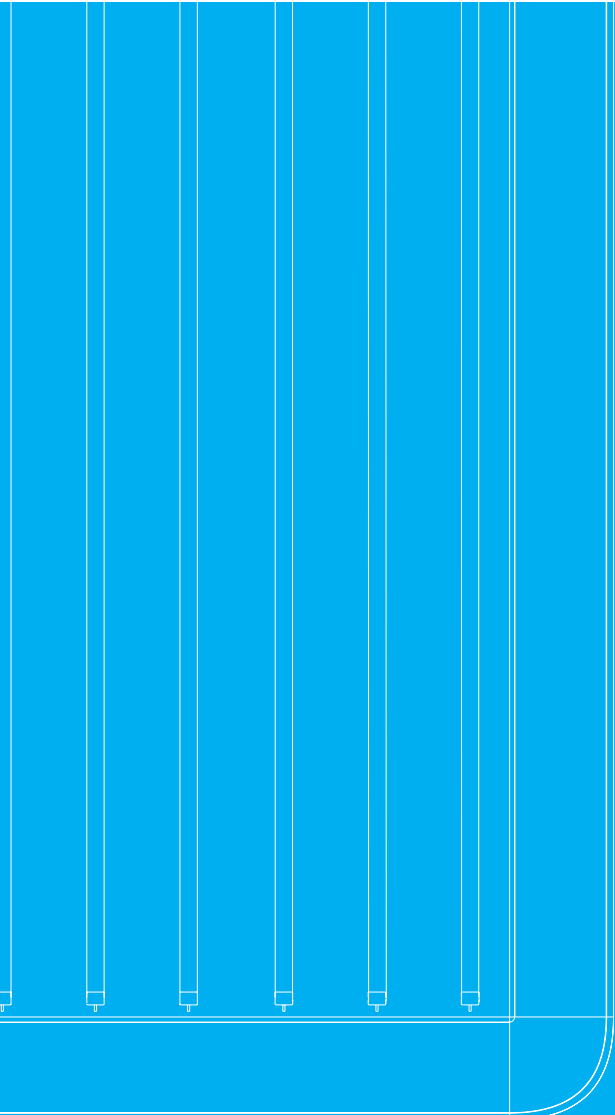

Facts: Denta / Denta plus

- High power – Denta 4,000 lux /Dentaplus 5,800 lux.
- Environmentally friendly and energy-saving.
- Dimmer switch and indirect light (upward lighting).
- Size: 1,400 x 700 cm.
- Weight: 11 kg.

D-TEC AB

Varavägen 12

SE-465 94 Nossebro

Phone: +46 (0)31-96 93 02 Fax: +46 (0)31-96 60 92

E-mail: mailbox@d-tec.se

www.d-tec.se